

MANI

THE BEST QUALITY IN THE WORLD,
TO THE WORLD.

MANI TROCAR KIT




MANI TROCAR KIT


Ultimate Sharpness, Minimal Stress

TROCAR S

Expanded product line.
Individually packed items reduce waste
and are ideal as a backup.

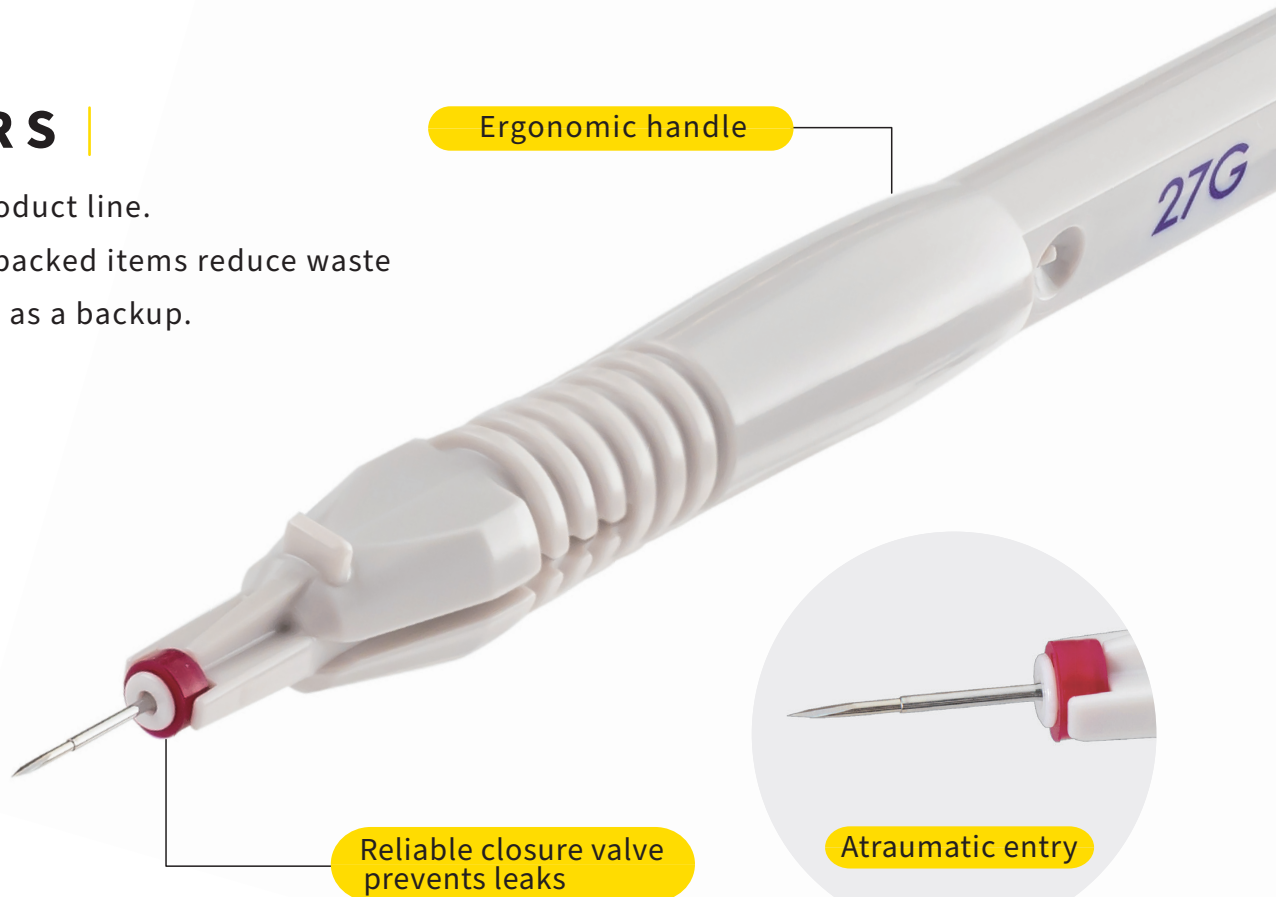

INFUSION CANNULA S

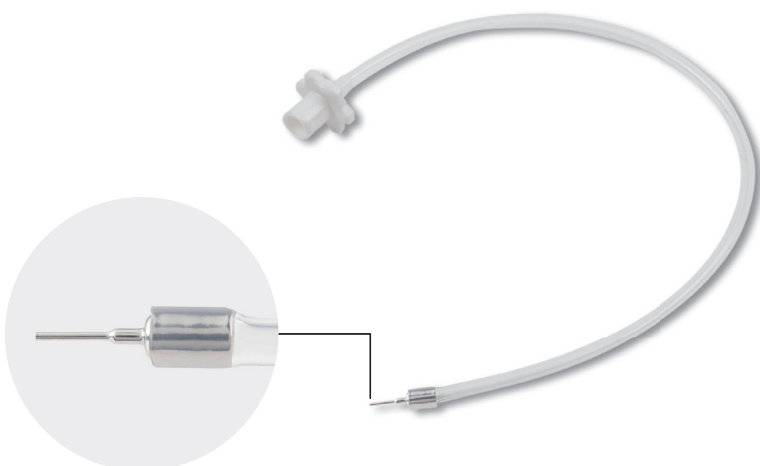

**Easily connected without
valve removal**





Inserts directly into port



Connection status

* Confirm the absence of any gap.

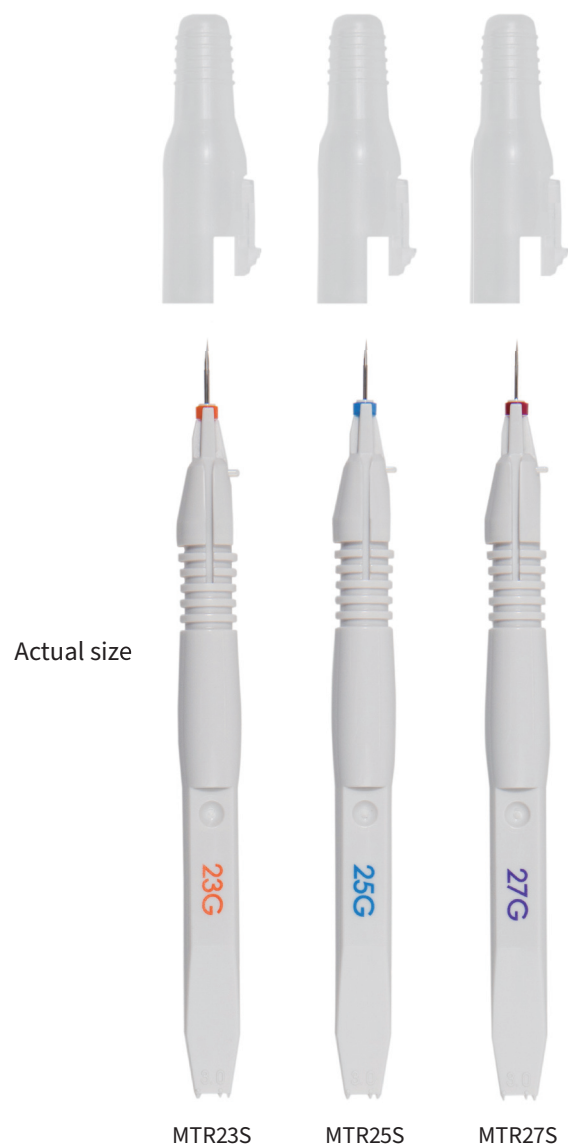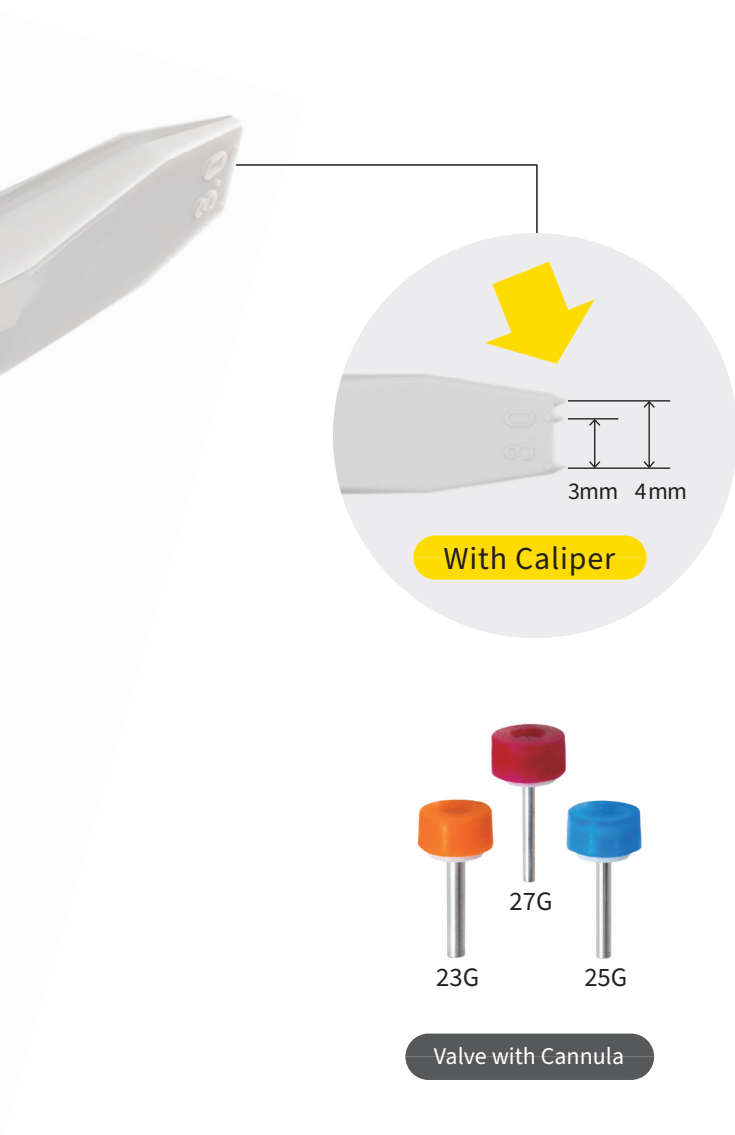


TROCAR KIT S



1 Surgery Case (1 Pack)

- TROCAR S × 3 Pcs.
- INFUSION CANNULA S × 1 Pc.

* Individually packaged trocars are available for use as a fourth port or as a backup.

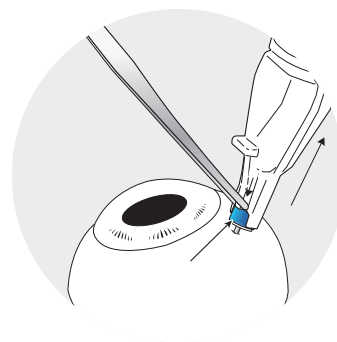

When holding the cannula

This product can be inserted into the sclera without holding the valved cannula. Withdraw the trocar handle after scleral placement. If the valved cannula does not detach easily, withdraw the trocar handle while gently holding the cannula with fine forceps. Take care not to touch the stop at the end of the handle as shown in the figure at right then position the cannula.

MANI TROCAR KIT

TROCAR KIT S

3 Kits/ Box

Description	Color Coding	Order No.
TROCAR KIT 27G S		MTK27S
TROCAR KIT 25G S		MTK25S
TROCAR KIT 23G S		MTK23S

TROCAR S

6 Pcs./ Box

Description	Color Coding	Order No.
TROCAR 27G S		MTR27S
TROCAR 25G S		MTR25S
TROCAR 23G S		MTR23S

* Individually packed – ideal for backup

INFUSION CANNULA S

6 Pcs./ Box

Description	Color Coding	Order No.
INFUSION CANNULA 27G S		MIC27S
INFUSION CANNULA 25G S		MIC25S
INFUSION CANNULA 23G S		MIC23S

* Individually packed – ideal for backup



MANI, INC.

8-3 KIYOHARA INDUSTRIAL PARK, UTSUNOMIYA,
TOCHIGI, 321-3231, JAPAN
Phone: +81-28-667-7565 / E-mail: seg-ovs@mani.inc /
URL: www.mani.co.jp/en/



Note : Design and specifications are subject to change without notice.

S0011